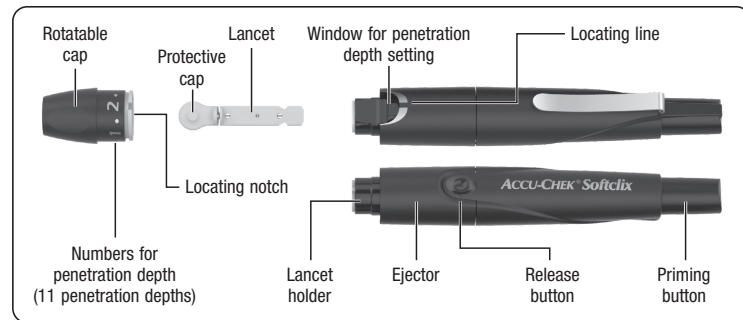
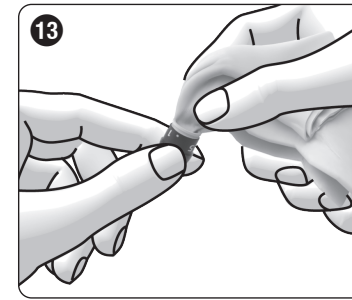
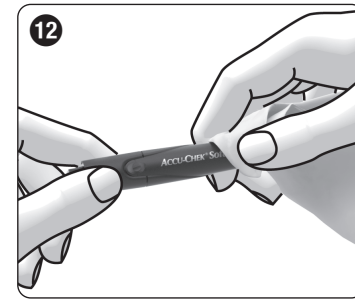
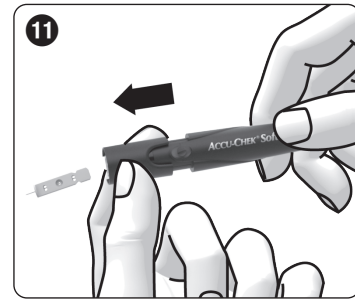
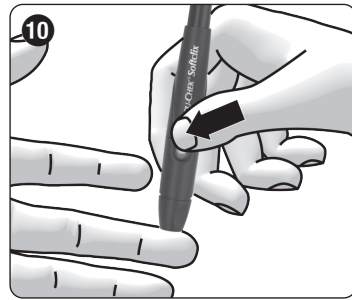
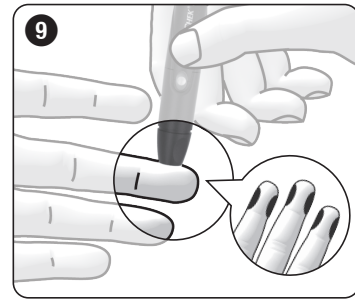
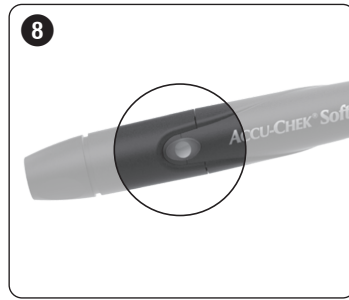
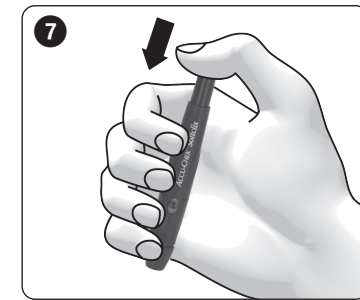
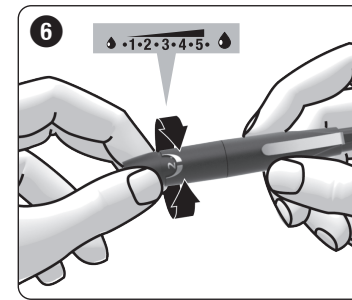
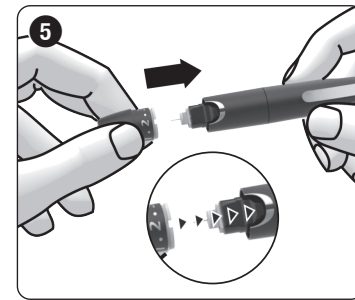
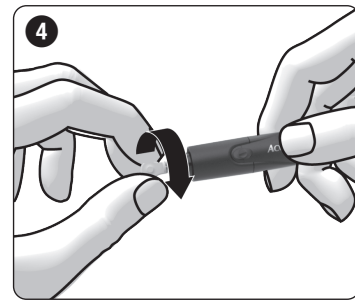
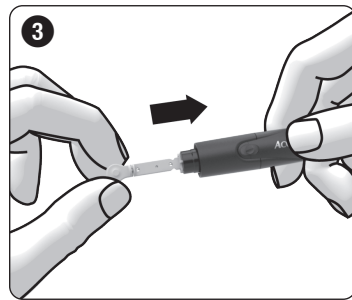
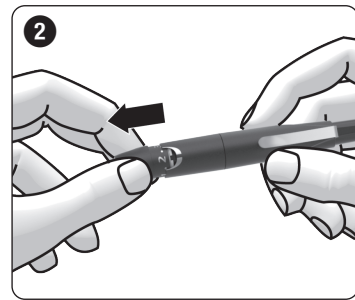
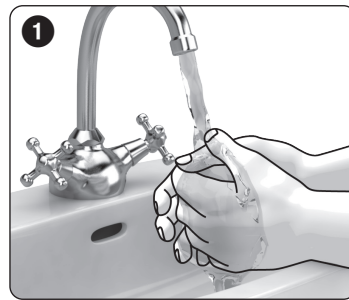
ACCU-CHEK® Softclix

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DE 09413375001(02) - US 09413375002

Risk of injury
Parts of the lancing device, for example, the priming button, may shoot away during use. Do not point the lancing device at your face or other people.

If the lancing device or lancet was dropped or exposed to other mechanical stress, check them for damages, for example, cracks. Do not use them if they are damaged.

Only insert Accu-Chek Softclix lancets into the Accu-Chek Softclix lancing device. Other lancets may impair the operation of the lancing device or damage it permanently.

Preparation

▶ Wash your hands with warm water and soap before you obtain blood (see 1). Dry your hands thoroughly. This reduces contamination of the puncture site and stimulates blood flow.

Inserting the Lancet

You have to insert a lancet to be able to obtain blood with the lancing device.

WARNING
Risk of infection
Lancets with missing protective cap are not sterile anymore. If the protective cap is missing, do not use the lancet.

▶ Remove the cap from the lancing device (see 2).
▶ Insert a new lancet into the lancet holder as far as it will go (see 3). You must hear the lancet click into place.
▶ Twist the protective cap off of the lancet (see 4).
▶ Place the cap back on again. The locating notch in the cap must be directly aligned with the locating line on the lancing device (see 5). You must hear the cap click into place.

Setting the Penetration Depth

You can set 11 penetration depths (0.5 to 5.5), so that you can adjust the penetration depth to the texture of your skin. The penetration depth set is shown in the window. The larger the number, the greater the penetration depth.

If you have no experience with this lancing device, we recommend setting a low penetration depth, such as penetration depth 2.

WARNING
Risk of injury
If the penetration depth is too large, the puncture can injure children. Before using the lancing device for the first time on children, ask your healthcare professional. Start with the lowest penetration depth.

▶ Rotate the cap until the desired penetration depth is displayed in the window (see 6).

Priming the Lancing Device

In order to be able to obtain blood, you must prime the lancing device. Only prime the lancing device just before obtaining blood.

▶ Press the priming button down as far as it will go (see 7). Do not press the release button simultaneously.

The lancing device is primed when the centre of the release button is yellow (see 8).

Obtaining a Blood Drop

WARNING
Risk of infection
The lancet was used several times to obtain blood. To prevent infections from being transmitted, use the lancet only once to obtain blood.

Risk of infection
In rare cases, the lancet is not pulled back into the lancing device after you obtained blood. To prevent infections from being transmitted, check that the lancet does not protrude from the cap after you obtained blood. If the lancet protrudes, do not touch the front of the cap. Remove the cap from the lancing device with care. Eject the lancet and dispose of it. Dispose of a used lancet in such a way as to prevent injury from the needle. For example, place the lancet into a cannula disposal container.

PRECAUTION
Risk of infection
The puncture site was not cleaned. Before obtaining blood clean the puncture site.

Risk of infection
The use by date of the lancet has expired. Use only lancets that are within the use by date. The use by date is printed next to the symbol on the packaging.

In general, blood can be obtained from any finger. Certain fingers can be unsuitable if, for example, a skin or fingernail infection is present. We recommend obtaining capillary blood from the sides of the fingertips as these areas are the least sensitive to pain. Use a new lancet each time you obtain blood so that obtaining blood remains virtually pain free.

Only use the lancing device when the cap is attached. Without the cap, the lancet penetrates too deeply and the puncture can become painful.

▶ Press the lancing device firmly against the selected puncture site (see 9).
▶ Press the release button (see 10).

The lancet is released and it penetrates the skin.

▶ Massage the finger in the direction of the fingertip to encourage a blood drop to form. The amount of blood that emerges depends on the penetration depth and the pressure used to hold the lancing device against the skin. If not enough blood emerges, apply more pressure on the lancing device when you next obtain blood. If that is not sufficient, increase the penetration depth progressively in addition. If too much blood emerges, decrease the penetration depth.

▶ Test your blood glucose immediately after you obtained blood, according to the instructions for use for your blood glucose meter.

We recommend protecting the puncture site from contamination afterwards.

Ejecting the Used Lancet

▶ Remove the cap from the lancing device (see 2).
▶ Slide the ejector forward (see 11). The used lancet is ejected.
▶ Place the cap back on again.
▶ Dispose of used lancets according to local regulations.

PRECAUTION
Risk of infection
The needle of a used lancet is not protected and can be touched accidentally. To prevent infections from being transmitted, dispose of a used lancet in such a way as to prevent injury from the needle. Place the lancet, for example, into a cannula disposal container.

Operating Conditions

Use the lancing device including accessories under the following operating conditions:

- Temperature range: +5 to +45 °C
- Relative humidity range: 5 to 90 %

Storing the Lancing Device and Lancets

Do not store the lancing device in a primed state. Do not store the lancing device and lancets at extreme temperatures, for example, in a hot car. This can impair the operation of the lancing device and lancets.

Store the lancing device only without a lancet inserted.

Cleaning and Disinfecting the Lancing Device and Cap

PRECAUTION
Risk of infection
The lancing device and cap may have come into contact with blood.

PRECAUTION
Risk of infection
The lancing device has come into contact with blood and is disposed of improperly. To prevent infections from being transmitted, dispose of the used lancing device as infectious material according to the regulations applicable in your country.

During obtaining a blood drop, the lancing device may come into contact with blood.

To prevent infections from being transmitted, clean and disinfect the lancing device and cap always before someone else handles them, for example, to assist you.

To prevent infections from being transmitted, you must clean and disinfect the lancing device and cap regularly:

- At least once a week
- When there is blood on the lancing device or cap
- Always before someone else handles the lancing device, for example, to assist you.

You need lint-free cloths in which you can wrap the whole lancing device, water and 70 % isopropanol.

Before you disinfect the lancing device and the cap, you first have to clean them to remove blood and other dirt.

To avoid damaging the lancing device and the cap:

- Do not dip the lancing device and the cap in the fluids.
- Use only water for cleaning and only 70 % isopropanol for disinfecting.
- ▶ Remove the cap from the lancing device (see 2).
- ▶ Eject the lancet (see 11).

Cleaning

▶ Moisten a cloth with water.

▶ Wipe the lancing device and the cap thoroughly. Wipe the lancing device from the outside only (see 12) and the cap from the outside and the inside (see 13).

▶ Dry the lancing device and the cap with a dry cloth.

Disinfecting

▶ Dampen a cloth with 70 % isopropanol.

▶ Wipe the lancing device and the cap thoroughly for 2 minutes each. Wipe the lancing device from the outside only and the cap from the outside and the inside.

▶ Place the cap back on again.

▶ Wrap the whole lancing device in the cloth.

▶ Leave the lancing device wrapped for 8 minutes.

▶ Unwrap the lancing device and let it dry in the air.

Disposing of the Lancing Device

PRECAUTION
Risk of infection
The lancing device has come into contact with blood and is disposed of improperly. To prevent infections from being transmitted, dispose of the used lancing device as infectious material according to the regulations applicable in your country.

During obtaining a blood drop, the lancing device may come into contact with blood.

A used lancing device therefore carries a risk of infection.

Dispose of an unused lancing device as plastic waste.

For information about correct disposal, contact your local council or authority.

Alternative Sites for Obtaining Blood

You can obtain capillary blood for blood glucose testing not only from the fingertips, but also from other sites (palm, forearm, upper arm, calf or thigh), so-called alternative site testing (abbreviated as AST). To be able to obtain capillary blood from other sites, you have to replace the regular cap by a special cap, the Accu-Chek Softclix AST cap.

Ordering Information

If you want to order an Accu-Chek Softclix lancing device, Accu-Chek Softclix lancets or an Accu-Chek Softclix AST cap, contact customer support (see section *Customer Support*).

Reporting of Serious Incidents

For a patient/user/third party in the European Union and in countries with identical regulatory regime; if, during the use of this device or as a result of its use, a serious incident has occurred, please report it to the manufacturer and to your national authority.

Last Update

2021-03

Customer Support

Canada
Accu-Chek Customer Care: 1-800-363-7949
www.accu-chek.ca

EN Instructions for Use

Intended Use

Lancing Device

The Accu-Chek Softclix lancing device is a device for simple, virtually pain-free collection of capillary blood from the fingertips. The Accu-Chek Softclix lancing device is intended for single patient use only! It is only allowed to be used to obtain blood from one person.

There is a risk of infections being transmitted if the lancing device is used by other people, even by family members, or if healthcare professionals use this lancing device to obtain blood from different people. This lancing device is therefore not suitable for professional use in healthcare facilities and institutions. It is not a sterile product.

People in the area of diabetes self-monitoring are considered as target group.

Lancet

The Accu-Chek Softclix lancet and the Accu-Chek Softclix XL lancet are sterile single-lancets and are intended to be used with the Accu-Chek Softclix lancing device for simple, virtually pain-free collection of capillary blood. The lancet is a disposable article intended to be used only once each.

Each lancet has a protective cap that guarantees sterile condition until its first use. The lancets are sterilized using irradiation.

Before You Get Started

Read these instructions for use before you obtain a blood drop from your fingertip.

Information describing a situation for which there is a foreseeable serious hazard with the use of the device, is preceded by the following heading:

WARNING

Information regarding any special care you should exercise for the safe and effective use of the device, or to avoid damage to the device

that could occur as a result of use, including misuse, is preceded by the following heading:

PRECAUTION

WARNING

Risk of suffocation

This product contains small parts that can be swallowed.

Keep this product away from people who might swallow small parts, for example, small children.

Risk of infection

During obtaining a blood drop, the lancing device may come into contact with blood.

To prevent infections from being transmitted, the lancing device must not be used by other people, even by family members. Healthcare professionals must not use the lancing device to obtain blood from different people.

Risk of infection

The lancing device or lancet has come into contact with blood and has protruding sharps due to damage.

To prevent infections from being transmitted, check the lancing device and lancet for damages. Dispose of a damaged lancing device as infectious material according to the regulations applicable in your country. Dispose of a damaged lancet in such a way as to prevent injury from the needle. For example, place the lancet into a cannula disposal container.

PRECAUTION

Risk of injury

Skin contact to the lancing device or lancet may cause skin irritation.

If your skin becomes irritated or inflamed, consult your healthcare professional.

Risk of injury

The needle of the lancet is damaged due to use on a hard surface.

Only use the lancet to obtain a blood drop.

Consult instructions for use or consult electronic instructions for use

Caution, refer to safety-related notes in the instructions for use accompanying this product.

Use by

Sterilized using irradiation

Use only once

Single patient multiple use

Medical device

Manufacturer

Unique device identifier

Catalogue number

Batch code

Complies with the provisions of the applicable EU Legislation

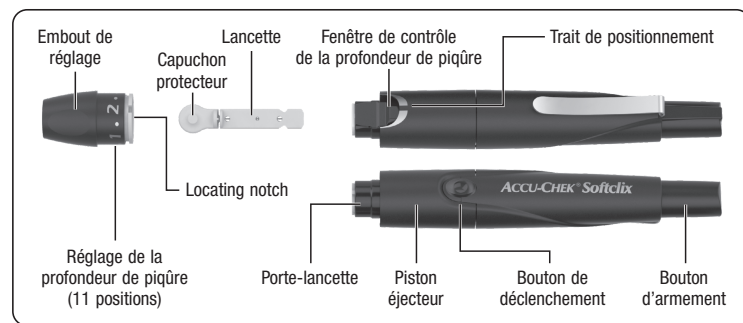
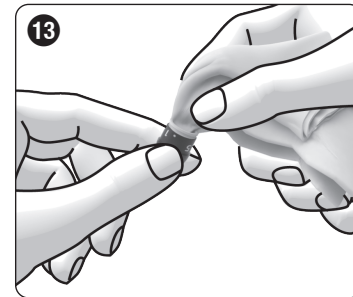
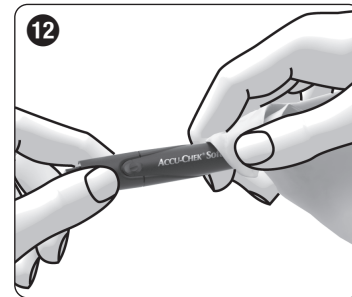
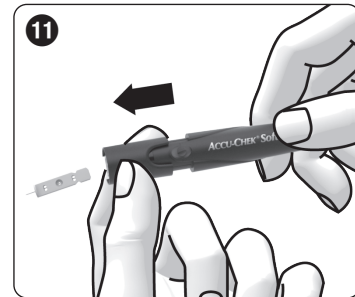
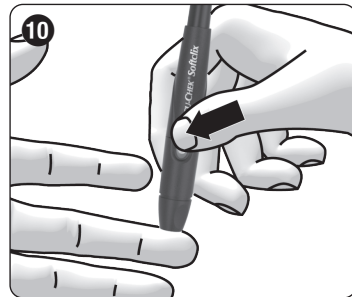
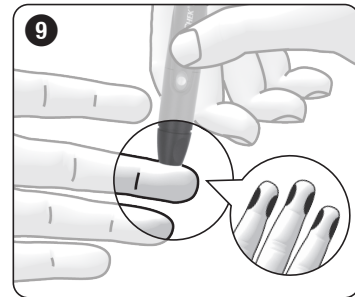
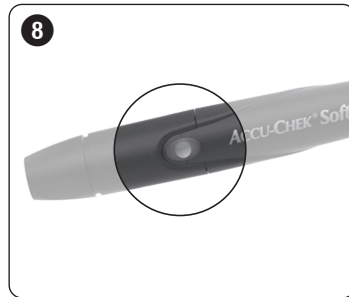
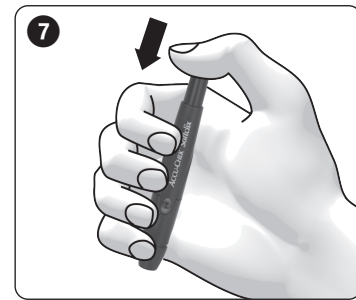
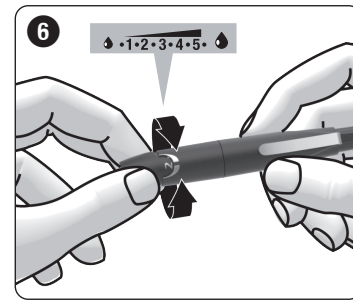
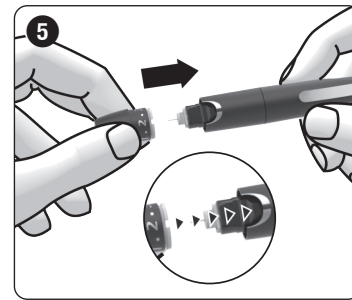
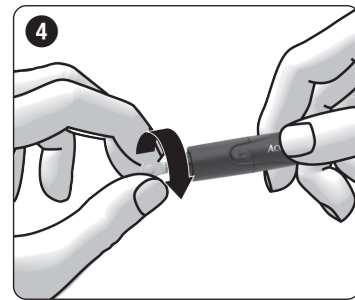
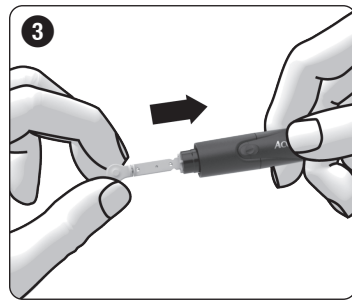
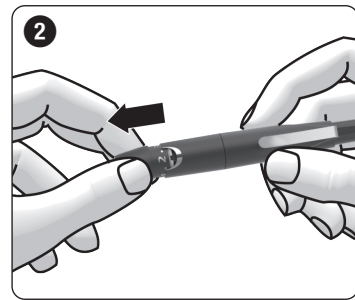
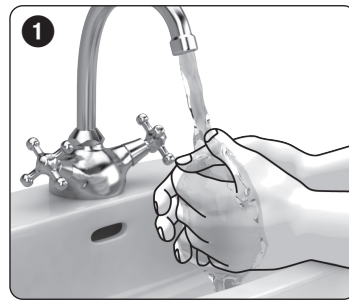
ACCU-CHEK and SOFTCLIX are trademarks of Roche.

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© 2021 Roche Diabetes Care GmbH
Roche Diabetes Care GmbH