
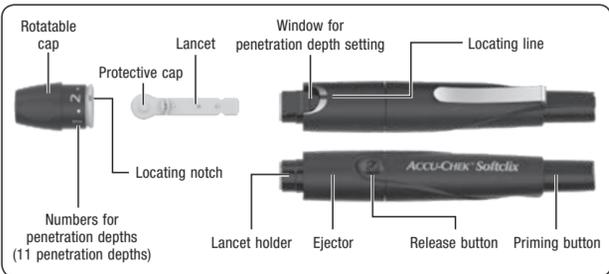
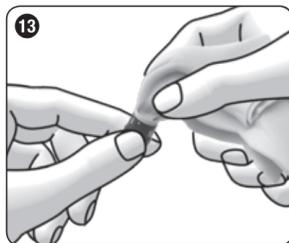
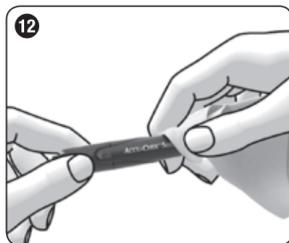
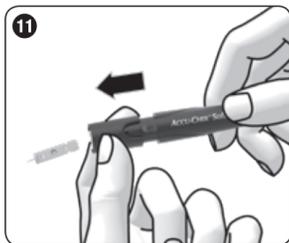
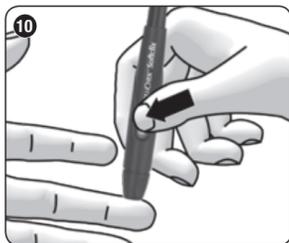
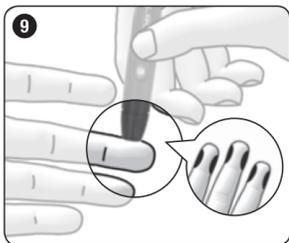
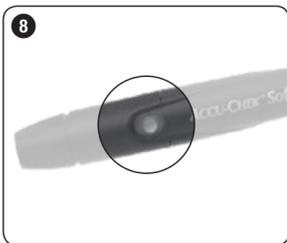
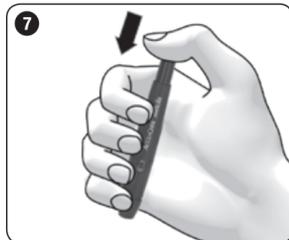
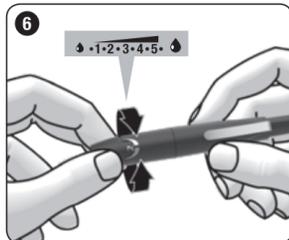
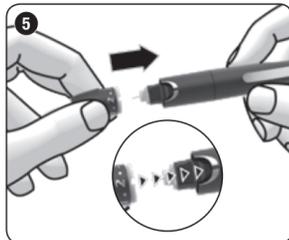
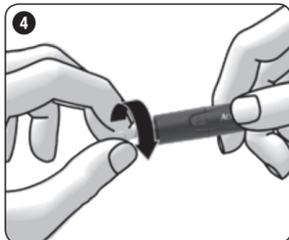
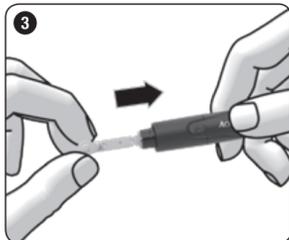
ACCU-CHEK® Softclix

Lancing device

Autopiqueur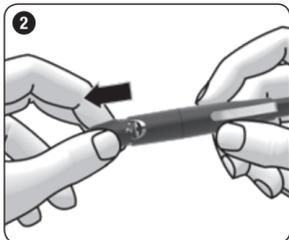

EN Instructions for use

The Accu-Chek Softclix lancing device is a device for simple, virtually pain-free collection of capillary blood from the fingertips. The lancing device has 11 different penetration depths so that you can adjust the penetration depth to the texture of your skin.

These instructions for use feature the following 2 symbols:

This symbol indicates a **possible risk of injury or of damage to your health**.

This symbol draws attention to actions that could result in **damage to the lancing device**.

• The **Accu-Chek Softclix lancing device is intended for personal use only!**

It is only allowed to be used to obtain blood from **one** person.

There is a risk of infections being transmitted if the lancing device is used by other people, even by family members, or if healthcare professionals use this lancing device to obtain blood from different people. This lancing device is therefore not suitable for professional use in healthcare facilities and institutions.

• Keep the lancing device with all its components out of reach of children under 3 years. There is a risk of suffocation if small parts (e.g. caps or similar objects) are swallowed.

• If the lancing device falls with a lancet inserted, the lancet may become loose in the lancet holder. You cannot use it anymore to obtain blood. In rare cases, the lancet could protrude from the opening in the cap and you could injure yourself on it. Therefore, do not touch the front of the cap. If the lancing device falls, pick it up with care. Remove the cap from the lancing device. Pay attention to the lancet so that you do not injure yourself on it. Always eject the lancet and discard it.

Only use Accu-Chek Softclix lancets in the Accu-Chek Softclix lancing device. Other lancets may impair the operation of the lancing device or damage the lancing device permanently.

Preparation

► Wash your hands with warm water and soap before you obtain blood (see **1**). Dry your hands thoroughly. This reduces contamination of the puncture site and stimulates blood flow.

Inserting the lancet

You have to insert a lancet to be able to obtain blood with the lancing device.

► Remove the cap from the lancing device (see **2**).

► Insert a new lancet into the lancet holder as far as it will go (see **3**). The lancet must audibly click into place.

► Twist the protective cap off of the lancet (see **4**).

► Place the cap back on again. The locating notch in the cap must be directly aligned with the locating line on the lancing device (see **5**). The cap must audibly click into place.

Setting the penetration depth

You can set 11 penetration depths (0,5 to 5,5). The penetration depth set is shown in the window. The larger the number, the greater the penetration depth.

If you have no experience with this lancing device, we recommend setting a low penetration depth, such as penetration depth 2.

► Rotate the cap until the desired penetration depth is set (see **6**).

Priming the lancing device

In order to be able to obtain blood, you must prime the lancing device.

► Press the priming button down as far as it will go (see **7**). Do not press the release button simultaneously.

The lancing device is primed when the centre of the release button is yellow (see **8**).

Only prime the lancing device just before obtaining blood.

Obtaining a blood drop

In general, blood can be obtained from any finger. Certain fingers can be unsuitable if, for example, a skin or fingernail infection is present. We recommend obtaining capillary blood from the sides of the fingertips as these areas are the least sensitive to pain.

• Always use a new lancet when you obtain blood. In doing this, you reduce the risk of infection, and obtaining blood remains virtually pain free.

• **Only** use the lancing device with the cap on. Without the cap, the lancet penetrates too deeply and the puncture can become painful.

► Press the lancing device firmly against the selected puncture site (see **9**).

► Press the release button (see **10**).

The lancet is released and it penetrates the skin.

► Massage the finger in the direction of the fingertip to encourage a blood drop to form.

The amount of blood that emerges depends on the penetration depth and the pressure used to hold the lancing device against the skin.

If not enough blood emerges, apply more pressure on the lancing device when you next obtain blood. If that is not sufficient, increase the penetration depth progressively in addition.

If too much blood emerges, decrease the penetration depth.

► Test your blood glucose immediately after you obtained blood, according to the instructions for use for your blood glucose meter.

We recommend protecting the puncture site from contamination afterwards.

Ejecting the used lancet

► Remove the cap from the lancing device (see **2**).

► Slide the ejector forward (see **11**).

The used lancet is ejected.

► Place the cap back on again.

► Dispose of used lancets according to local regulations.

Dispose of used lancets in such a way as to prevent injury from the needles.

Storing the lancing device

Do not store the lancing device in a primed state or at extreme temperatures, for example, in a hot car. This can impair the operation of the lancing device.

Store the lancing device only without a lancet inserted.

Cleaning and disinfecting the lancing device and cap

To prevent infections from being transmitted, you must clean and disinfect the lancing device and cap regularly:

- At least once a week
- When there is blood on the lancing device or cap
- Always before someone else handles the lancing device, for example, to assist you.

You need lint-free cloths in which you can wrap the whole lancing device, water and 70 % isopropanol.

Before you disinfect the lancing device and the cap, you first have to clean them to remove blood and other dirt.

To avoid damaging the lancing device and the cap:

- Do not dip the lancing device and the cap in the fluids.
- Use only water for cleaning and only 70 % isopropanol for disinfecting.

► Remove the cap from the lancing device (see **2**).

► Eject the lancet (see **11**).

Cleaning

► Moisten a cloth with water.

► Wipe the lancing device and the cap thoroughly; the lancing device from the outside only (see **12**) and the cap from the outside and the inside (see **13**).

► Dry the lancing device and the cap with a dry cloth.

Disinfecting

► Dampen a cloth with 70 % isopropanol.

► Wipe the lancing device and the cap thoroughly for 2 minutes each; the lancing device from the outside only and the cap from the outside and the inside.

► Place the cap back on again.

► Wrap the whole lancing device in the cloth.

► Leave the lancing device wrapped for 8 minutes.

► Unwrap the lancing device and let it dry in the air.

Alternative sites for obtaining blood

You can obtain capillary blood for blood glucose testing not only from the fingertips, but also from other sites (palm, forearm, upper arm, calf or thigh), so-called alternative site testing (abbreviated as AST). To be able to obtain capillary blood from other sites, you have to replace the regular cap by a special cap, the Accu-Chek Softclix AST cap.

Ordering information

If you want to order an Accu-Chek Softclix lancing device, Accu-Chek Softclix lancets or an Accu-Chek Softclix AST cap, please contact your customer support and service centre (see section "Customer Support and Service Centre").

- at least 1 time per week,
- if there are traces of blood,
- before every manipulation of the lancing device by someone else, for example, to assist you.

You must use disinfecting cloths in which you can wrap the whole lancing device, water and 70 % isopropanol.

Before you disinfect the lancing device and the cap, you first have to clean them to remove blood and other dirt.

To avoid damaging the lancing device and the cap:

- Do not dip the lancing device and the cap in the fluids.
- Use only water for cleaning and only 70 % isopropanol for disinfecting.

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© 2019 Roche Diabetes Care
ACCU-CHEK and SOFTCLIX are trademarks of Roche.