ACCU-CHEK®

ACCU-CHEK®
Guide

SIMPLY MORE


Monitor in the dark

The Accu-Chek Guide meter has a bright backlit display and a strip port light.

Helping you manage your diabetes

When it comes to diabetes, we believe in personalized treatment because we want you to get the most out of life.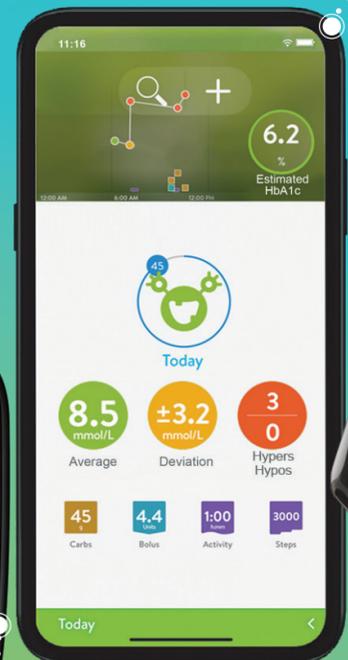

No-touch strip removal

With the strip ejector button, a single click is all it takes for fast, easy and hygienic removal of strips.

Spill-resistant

The vial is designed to hold your strips tightly in place, even upside down!

Widest** dosing area on the market

Apply a small drop of blood anywhere on the yellow edge of the strip.

Advanced accuracy

Results you can trust: Accu-Chek Guide strips exceed the current ISO standard.***

All your diabetes data in one place

Accu-Chek Guide syncs quickly with the mySugr® app.

Pair it once and your values will be transferred automatically from your meter into the app.

Plus, unlock mySugr PRO for FREE.*

Knowing your blood sugar levels and monitoring your data has never been so simple!

mySugr

How to start transferring your blood sugar levels automatically into the mySugr app

1. Download the mySugr app.

2. Open the app and select "Connections" from the menu.

3. Select "Accu-Chek Guide" and click on "Connect."

4. Turn on the Accu-Chek Guide meter and go to your "Settings."

5. Select "Wireless" and press "OK." Then select "Pairing" and press "OK." Then select "Pair Device" and press "OK."

6. Enter the code displayed on your meter into the notification on your smart device and confirm the pairing.

7. Return to "Wireless" in your meter settings and turn on "Auto-Send."

1. Average app rating in App Store and Google Play Store worldwide, 2023-03-01, 2023-05-31.
2. As of June 19, 2023.

ACCU-CHEK®
engage™

With Accu-Chek Guide, get exclusive access to the engage program:

> Unlock 24/7 diabetes support

> Redeem special offers

> Automatically activate your lifetime meter warranty†

Sign up today!

Go to engageProgram.ca or scan this code.

Need help signing up?


Contact Accu-Chek Customer Care: 1-800-363-7949
Mon.- Fri., 9 a.m.-6 p.m. (ET)

†As part of the Lifetime Warranty, you will receive support and assistance in the use of your Accu-Chek products throughout their lifetime. In the event of a defect or problem during normal use, we will provide you with a replacement product or a newer model for the lifetime of the product.

Roche

SIMPLY MORE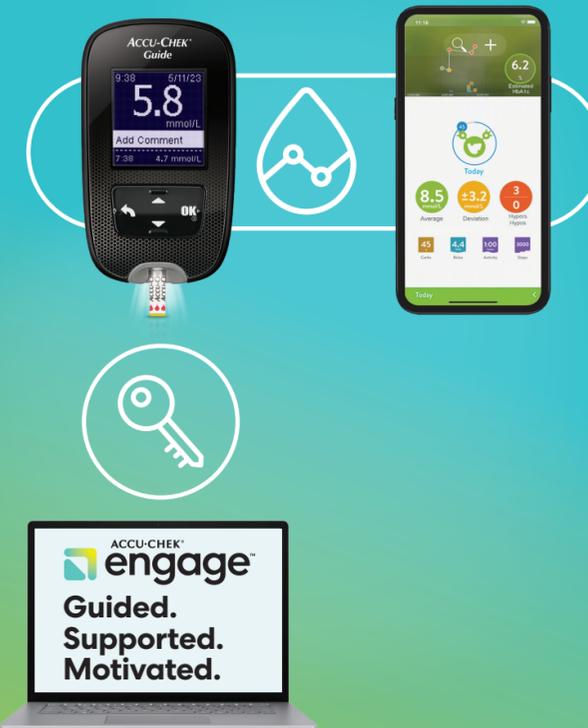

About Roche

Roche has been pioneering innovative diabetes technologies and solutions for more than 40 years. We offer a holistic, patient-centric approach for diabetes management, bringing true relief to people living with diabetes worldwide.

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*Téléchargez mySugr et profitez de plus de fonctionnalités avec mySugr PRO lorsque vous connectez et importez les données de votre glycomètre Accu-Chek Guide (valeur de 37,99 \$ par année). Afin de conserver l'accès à toutes les fonctions PRO, il vous faut synchroniser vos valeurs de glycémie au moins une fois tous les 30 jours. | ** Parmi les principaux fabricants de l'industrie. Données internes. | *** Une nouvelle génération de système de surveillance de la glycémie dépasse les normes internationales de précision (de l'anglais "New Generation Blood Glucose Monitoring System Exceeds International Accuracy Standards"); Ronald L. Brazg, M.D. et coll., Journal of Diabetes Science and Technology, 2016. | APPLE. Le logo APPLE et APP STORE sont des marques de commerce d'Apple Inc. Android, Google Play et le logo Google Play sont des marques de commerce de Google LLC. | ACCU-CHEK, ACCU-CHEK GUIDE et MYSUGR sont des marques de commerce de Roche. Tous droits réservés. © 2023 Roche Soins du diabète. CA-1815 10022833001