



# How sitesmart™ works


- 1 Select 4 sites where you want to inject.
- 2 Apply the 4 coloured stickers (located in the packaging) on those 4 sites on the body-map printed on the packaging.

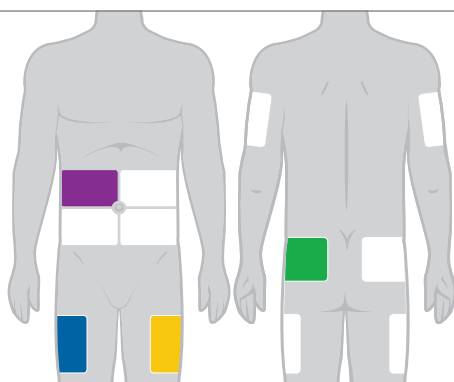

You can also scan this QR code (located on the packaging as well) to create an on-the-go plan.

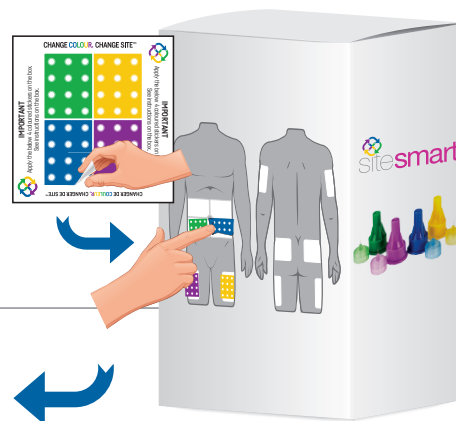

- 3 You randomly pick a coloured pen needle and inject in the corresponding site.

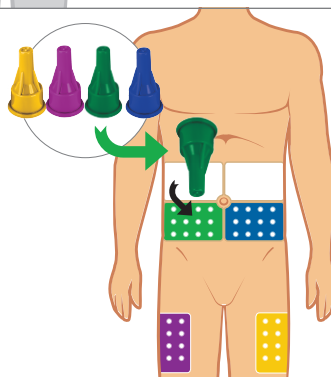

 **sitesmart™**

For insulin users.

sitesmart™ is a montméd product distributed by Roche Diabetes Care.  
sitesmart™ is a trademark of montméd Inc.  
© 2022 Roche Diabetes Care. All rights reserved. CA-1459

 **mont  
méd**  
Logistique Médicaments Canadienne  
Canadian Medical Logistics

 **Roche**