CONNECT HERE


Downloading the App

ON YOUR MOBILE DEVICE\*

- 1 Open the app store.
- 2 Find the ACCU-CHEK Connect diabetes management app.
- 3 Download and install the app.

\*Visit [accu-chek.ca/dmapp](http://accu-chek.ca/dmapp) for a list of compatible mobile devices.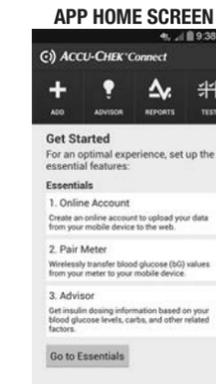


**WARNING**  
Do not share the use of the app with any other user.

Setting up an Online Account

ON YOUR MOBILE DEVICE

The following instructions are to set up your online account via your app; however, you can do this on your PC browser at [www.accu-chekconnect.com](http://www.accu-chekconnect.com)

- 1 Within the app, go to **Settings > Online Account**.
- 2 Select [Click HERE to Create an Online Account](#)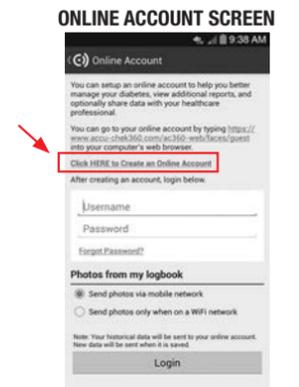


- 3 Add your account information.  
-Username must contain 5–30 characters (letters or numbers only).  
-Password must contain at least 8 characters (both letters and numbers required).

- 4 Select **No** to transfer your existing data unless you are on your PC and are an existing ACCU-CHEK 360<sup>®</sup> diabetes management system user.

- 5 Complete your diabetes-related information.
- 6 Close the browser by selecting the device **Home** button.

BACK BUTTON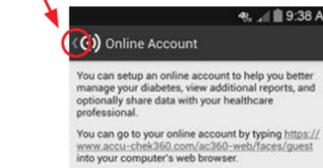


Your app and your online account are now connected for data transfer.

Pairing the Meter and Mobile Device


ON YOUR MOBILE DEVICE

- 1 Within the app, select **Settings/Essentials > Pair Meter**.
- 2 If prompted, ensure **Bluetooth<sup>®</sup>** wireless technology is on.
- 3 Select **Next** to enable **Auto-Send** and **Sync Time**.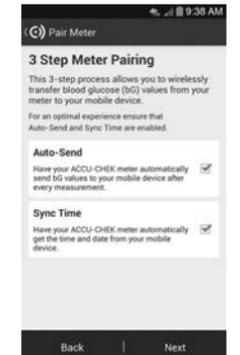

ON YOUR METER


- 4 Turn the meter on and use **OK** or **OK** to navigate and **OK** to select.
- 5 Select **Settings > Wireless > Yes**.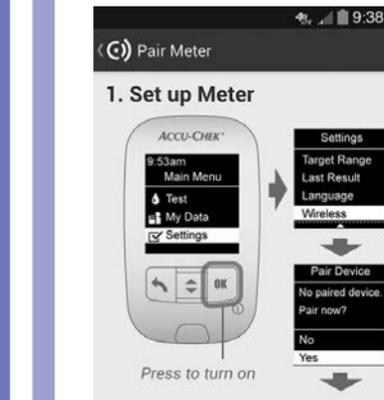

**NOTE:** The code which appears is valid for 90 seconds.

ON YOUR MOBILE DEVICE

- 6 When the code appears on the meter display, select **Next** in the app.
- 7 Select **Accu-Chek**. (Wait for pop-up menu to open.)
- 8 Enter the code from the meter display into the app.
- 9 Close the keyboard and select **OK** to confirm the code.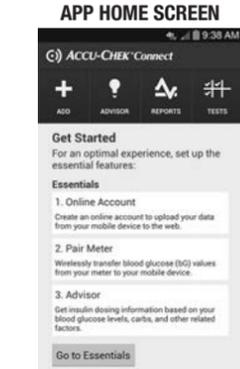


- 10 Wait for pairing process to complete.
- 11 Select **Close** or **Register Meter** after **Pairing successful** appears.
- 12 Select **Back** to return to the Home screen.

**NOTE:** After pairing, you may register your meter directly from the app.

Sending a Blood Glucose Result to Your Mobile Device

ON YOUR METER

After pairing is complete, turn the meter off before you perform a blood glucose test. Once you perform a successful blood glucose test, follow these instructions to send the test result to your mobile device.

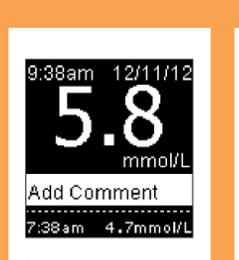
- 1 Press **OK** to add a comment to the test result.

- 2 Press **OK** or **OK** to highlight the desired comment and **OK** to select it. Press **OK** to send.

ON YOUR MOBILE DEVICE

- 3 Your blood glucose result appears on your app Home screen.

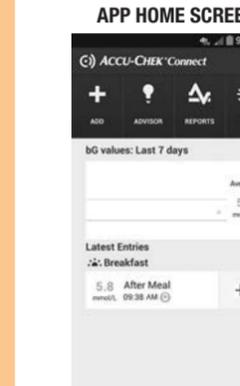The **Bluetooth<sup>®</sup>** word mark and logos are registered trademarks owned by Bluetooth SIG, Inc. and any use of such marks by Roche is under license.

ACCUCHEK, ACCUCHEK CONNECT, and ACCUCHEK 360<sup>®</sup> are trademarks of Roche.

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© 2015 Roche Diabetes Care
Roche Diabetes Care GmbH